


Quick Guide for RotoBed® AirSleep hybrid mattress

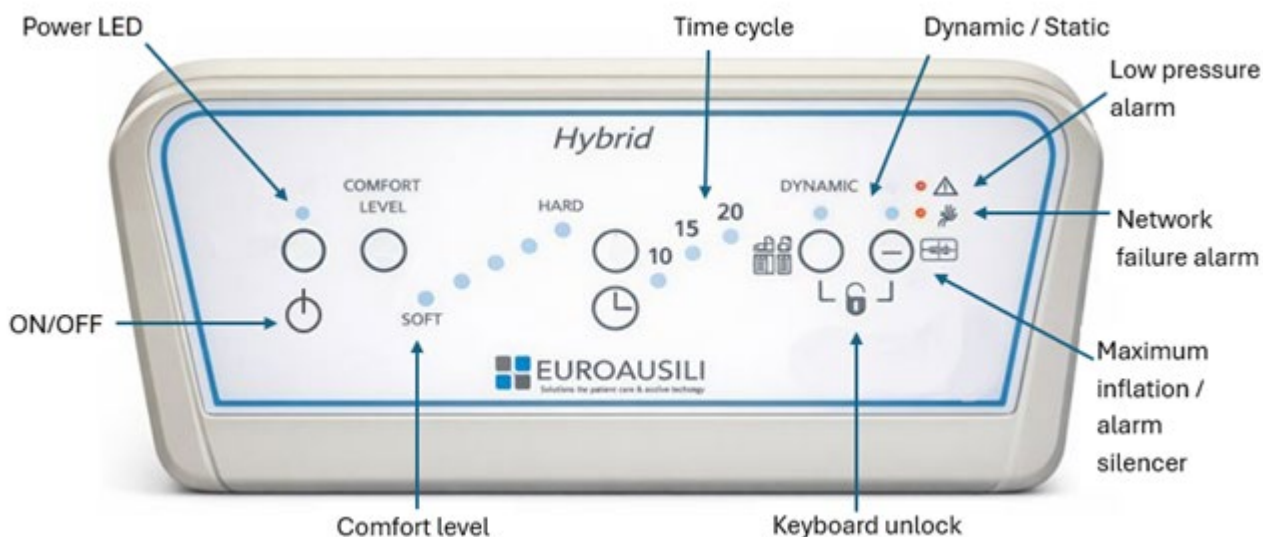

	Power on/off. When the compressor is plugged in but is turned off: the green light flashes. When the compressor is running: the green light is steady.
	<ul style="list-style-type: none"> • Low pressure alarm: red permanent LED and continuous sound to identify a malfunction, pressure drop or hose rupture. • Network error alarm: red flashing LED and intermittent sound
	Choose between 5 comfort levels. The blue LED indicates selected level from soft to hard.
	This button switches from dynamic to static mode and back. The activated blue LED indicates that dynamic mode is activated.
	Keyboard unlock: Press the 2 buttons at the same time. The keyboard will automatically lock after 2 minutes of inactivity to prevent accidental changes to the setup (all blue LEDs will be off, while the green LED will still be on).
	Button to turn off the alarm (this function is only active when the alarm is on) or to activate the maximum inflation mode to facilitate nursing procedures. The maximum inflation mode will automatically stop after 20 minutes, but the operator can manually switch it off at any time.

N.B. After 120 seconds without pressing any button, the keypad will automatically lock, and all blue LEDs will be turned off while the green LED will remain on. This solution is designed to make the compressor work with as little inconvenience as possible for patients and healthcare professionals. If the alarm is triggered while the keypad is locked, the corresponding red LED and alarm sound will be on, but it is possible to cancel the acoustic alarm without unlocking the control panel. Please note that the included mute button is only active when the alarm is active: as a result, there is no chance to set nursing mode without unlocking the keypad.



For access to full product description, please scan QR-code.

RotoBed®