
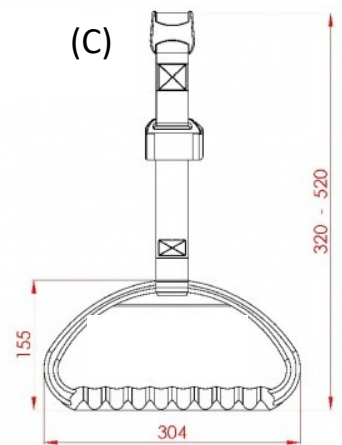
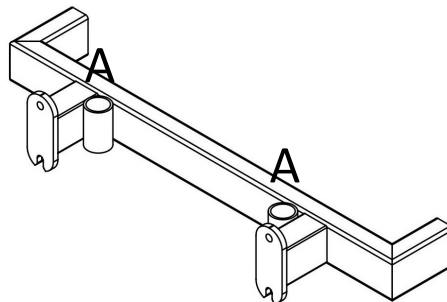



Instruction manual for Patient Lifting Pole

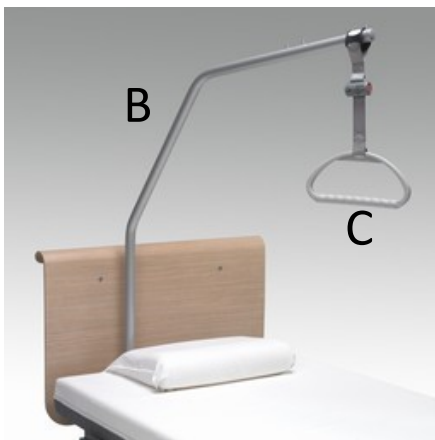

The holder for the Patient Lifting Pole (B) is located at the head end of the patient surface (A). From above, insert the Patient Lifting Pole into the holder (left- or righthand side) it is held by a bolt. Turn the Patient Lifting Pole, so that the bolt lies in one of the notches provided in the holder (Vertical or horizontal adjustment is possible). Place the Grib handle (C) between the hooks on the Patient Lifting Pole.

Attention! Grip handle and Belt consist of high quality plastic. However, all plastics are not unconditionally resistant to aging, therefore Grib handle and Belt can break or tear after long usage. This can cause injuries. Replace Grip handle including Belt after 4-5 years at the latest. The year of manufacture see label on the product.

Attention! Under high load, the Grip handle and Belt can break or tear and cause injuries. Only use the Patient Lifting Pole under a maximum load of 750 N (approx. 75 kg.)

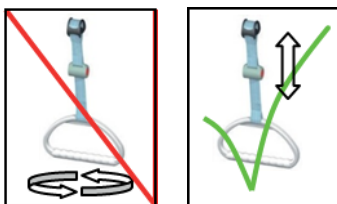

Attention!

The grip handle incl. belt must not be turned repeatedly on its vertical axis if hung up. This may cause damages of the automatic roll up.