

Extension of RotoBed®

RotoBed® can be extended in three ways in two lengths:

- Extension of the principal mattress + 10 cm.
- Extension of the foot mattress + 10 cm.
- Extension of both the principal mattress and the foot mattress + 20 cm.

Once the bed is fitted, lower it until it rests on the trolley (RotoBed®Trolley).

The head- and footboard can then be removed. Now mount the extension parts, both on the lying surface and on the frame before fitting the head- and footboard again. (See the pictures below).



Extension part for the frame + 10 cm.
Where either the principal mattress or the
foot mattress are extended.



Extension part for the frame + 20 cm.
Where both the principal mattress and the
foot mattress are extended.



Extension part for the lying surface + 10 cm.
(foot mattress).



Extension part for the lying surface + 10 cm.
(principal mattress).

Attention! After extending the main mattress, it is important to program the bed for a higher exit position to prevent the extension part from hitting the floor. Damage to the bed caused by this is not covered by the warranty. To avoid damage to the bed, the bed height must be programmed so that there is air between the bottom edge of the mattress and the floor when the bed is rotated to exit position (chair position).