



Mounting of RotoBed® Underbed Light

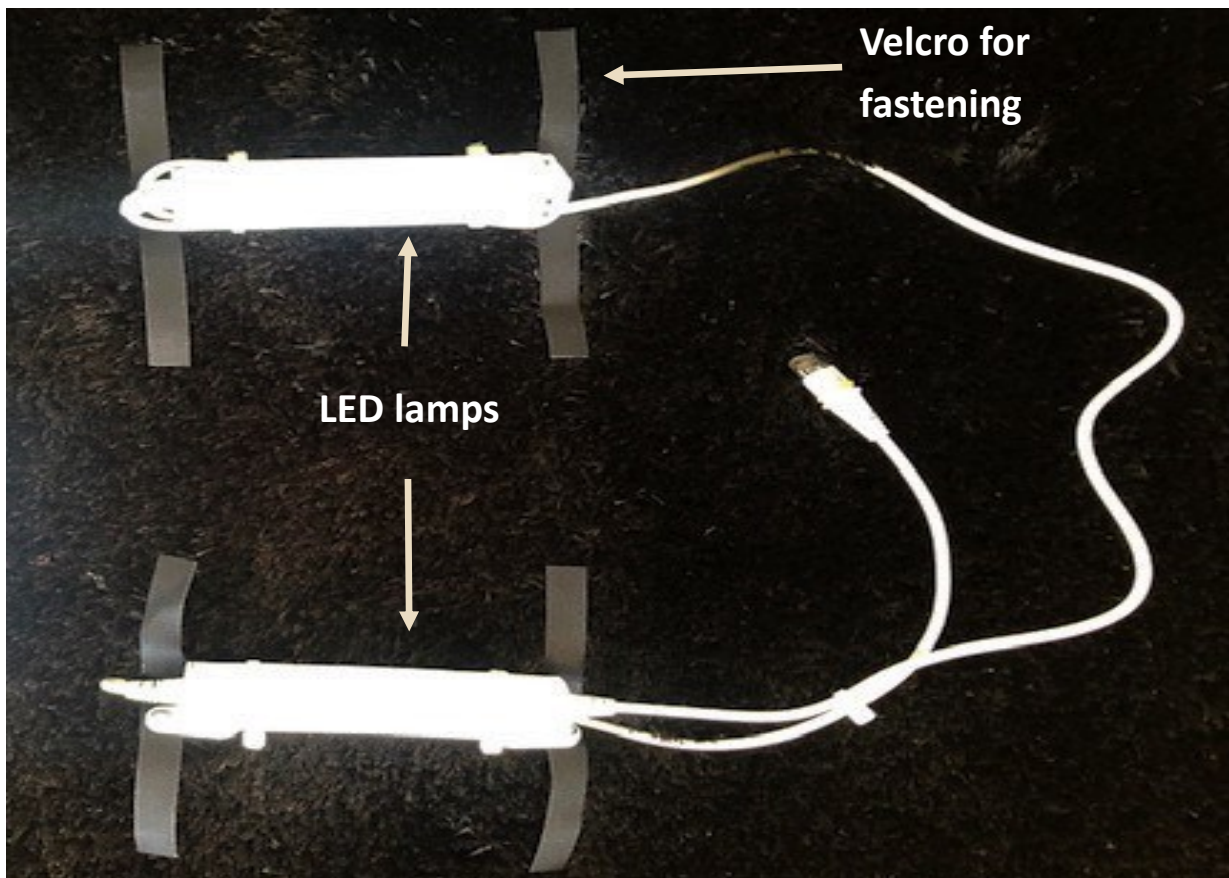


Start by removing the mattress. Move the bed to exit position and raise the leg-part to the highest possible. Now you are ready to mount the Underbed Light.

1. Removing the cover from the Signal box.

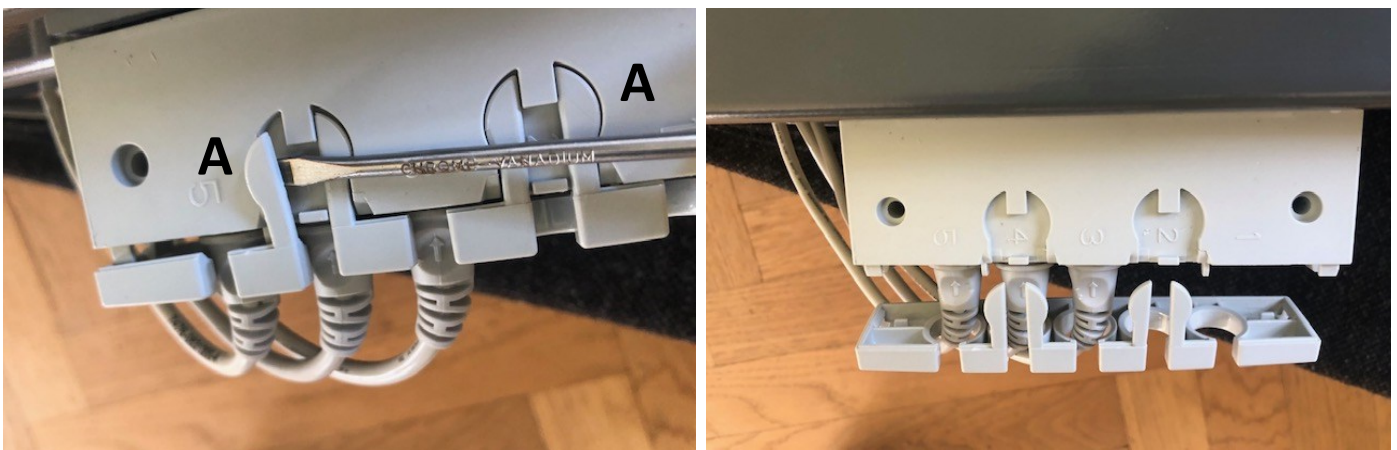


1.a) Use a flat screwdriver or similar. Wriggle to lift and push the 4 pins away (A) This way the cover of the Signal box will come off. The Signal box is now ready for installing the new cable from the LED lamp.

2. Mounting the LED lamps:

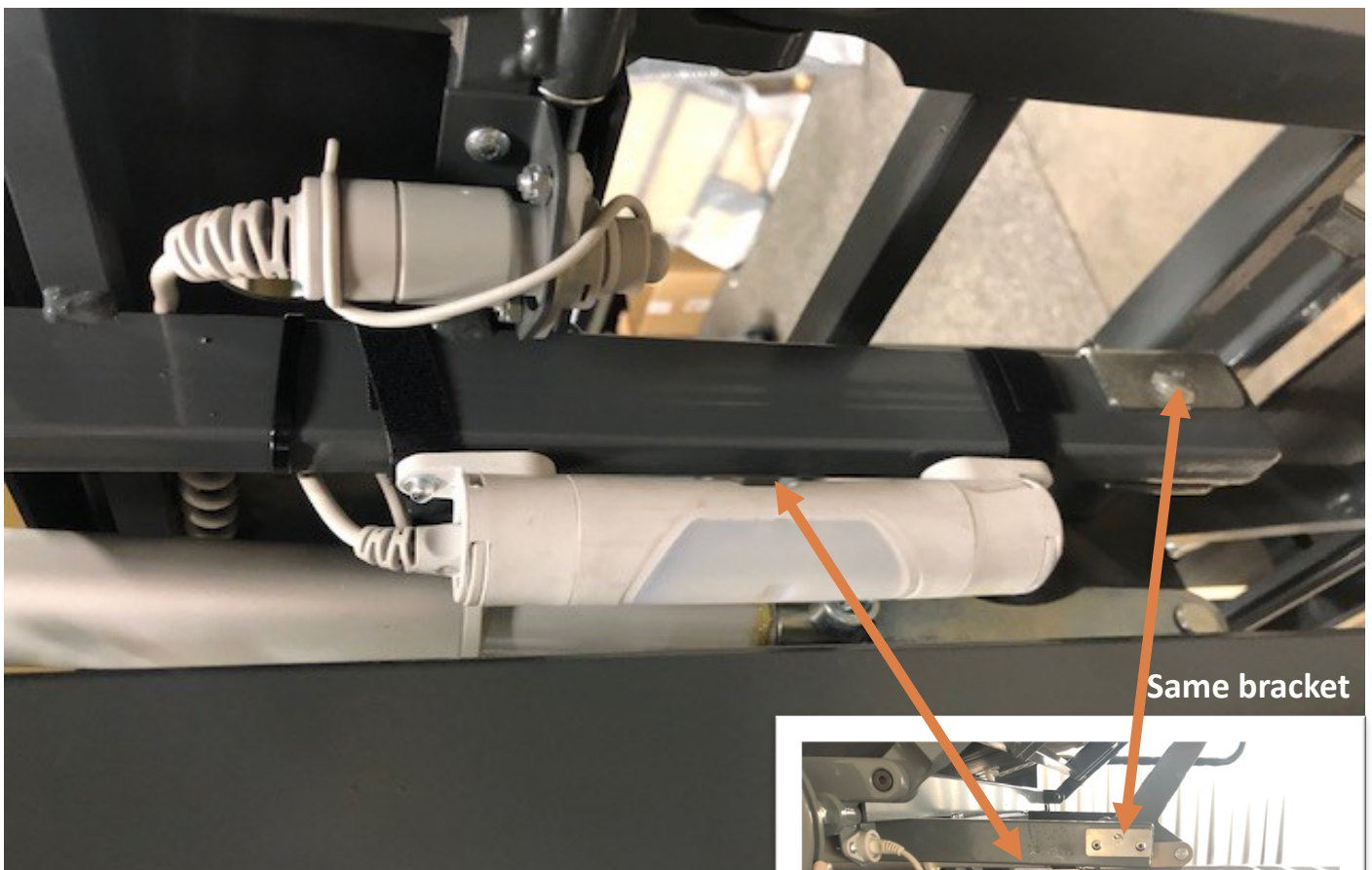

2.a) Use the velcro strips to fasten the LED lamps on the bedframe under the legpart on both sides of the bed.

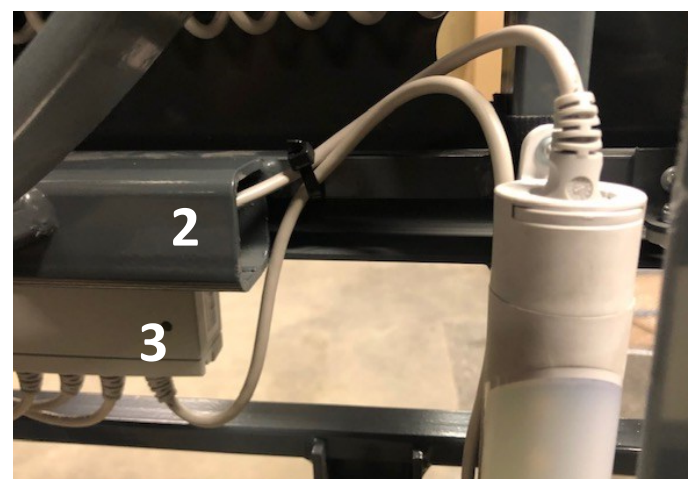

2.b) Take the long cable (1) connected to one of the LED lamps and place it inside the tube (2) and connect it to the other LED lamp. The short cable from LED lamp no. 2 must be plugged into the Signal box (3)