


Re-installation of axle with nylon wheels



U-Profile where the nylon wheel should run

Nylon wheel

If the 2 nylon wheels have fallen out of the U-Profile, it is not possible to move the leg-part

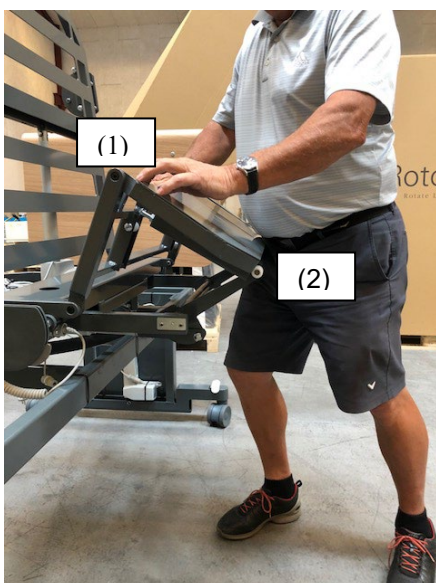

Start by raising the bed height to a convenient height. Lift up the upper leg-part (1) while pushing with your hip against the lower leg (2) part in direction backrest.

Replace the axel with the Nylon wheels in the U-Profile. Place your hands on the side of the foot-part and slide it carefully down - away from the backrest until it is in position (3)

NB! Watch out for your fingers and hands, that they are not pinched, the gas-spring on each side are quite powerful.